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
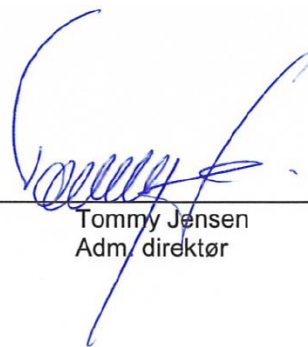
E-mail: info@ropox.dk

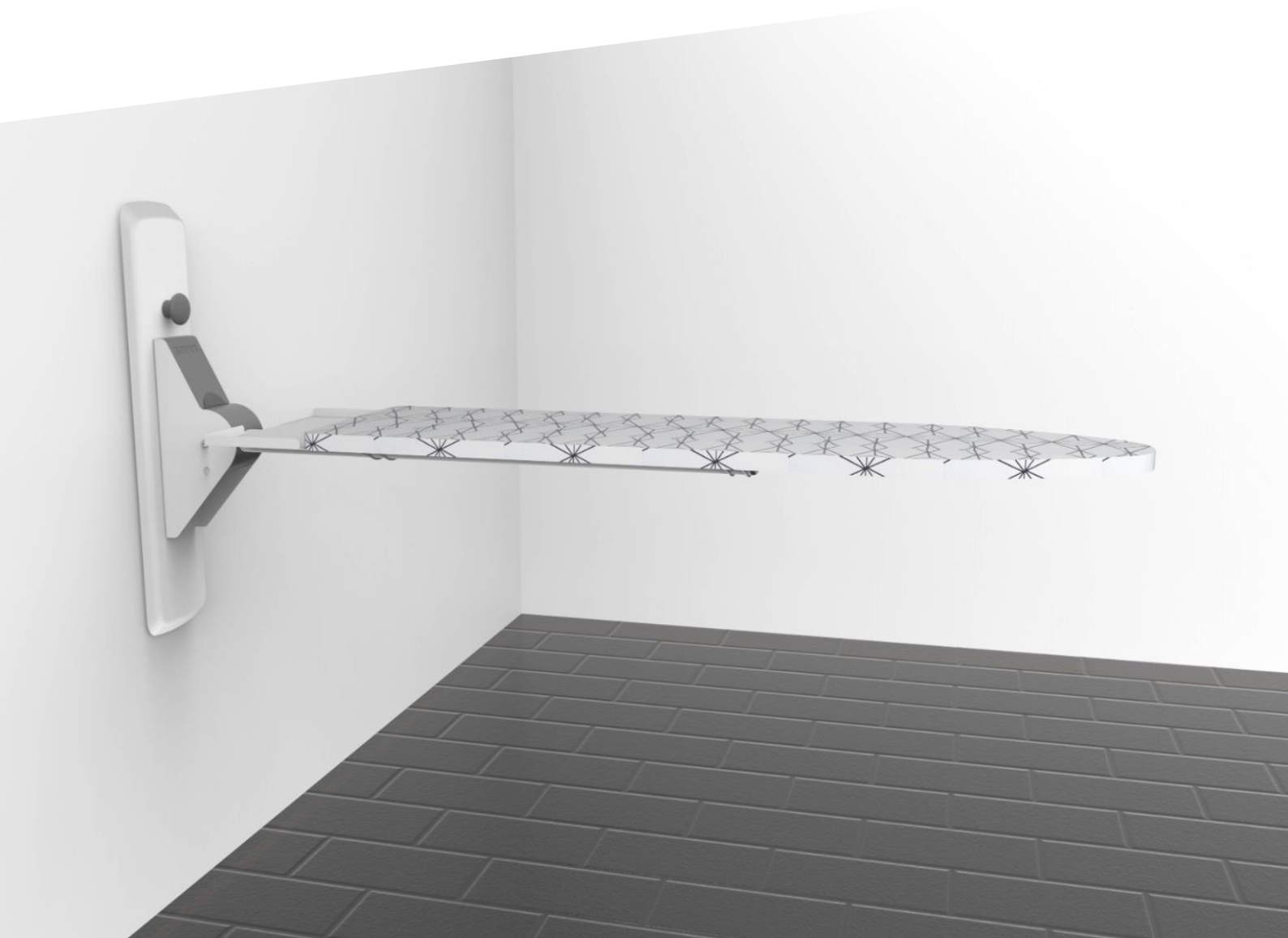www.ropox.dk

Ropox Ironing Board and Height Adjustment Unit for Ironing Board User Manual and Mounting Instructions

Keep this manual with the product at all times

Contents

CONTENTS	21
USER MANUAL	22
MOUNTING INSTRUCTIONS	29
SERVICE INFORMATION	31
SPARE PARTS	30
CE MARKING	36
COMPLAINTS	37

User manual

General product information

Ropox Ironing Board is a wall-mounted, fold-up ironing board ideal for wheelchair users. It may be combined with the Ropox Height Adjustment Unit for Ironing Board, which is available as an option.

Technical data

Product name:	Ropox Ironing Board	
Item number:	10-78500	Ropox Ironing Board
		
Accessory:	10-78510	Ropox Height Adjustment Unit for Ironing Board
Materials:	Steel Cover plates Aluminium Board	Surface-treated steel PA and ABS Surface-treated aluminium Melamine
Surface treatment:	Powder-coating Blue chromate	
Max. load:	According to DS/EN 12182 the max. load is 25kg (55lbs)	
Self-weight:	10-78500	4kg (8lbs 13oz)
	10-78510	3 kg (6lbs 10oz)
Application temperature:	10-40° C (50-104° F)	
Air humidity:	5-85 % (non-condensing)	
Manufacturer:	Ropox A/S DK-4700 Næstved, Tel: +45 55 75 05 00 E-mail: info@ropox.dk	

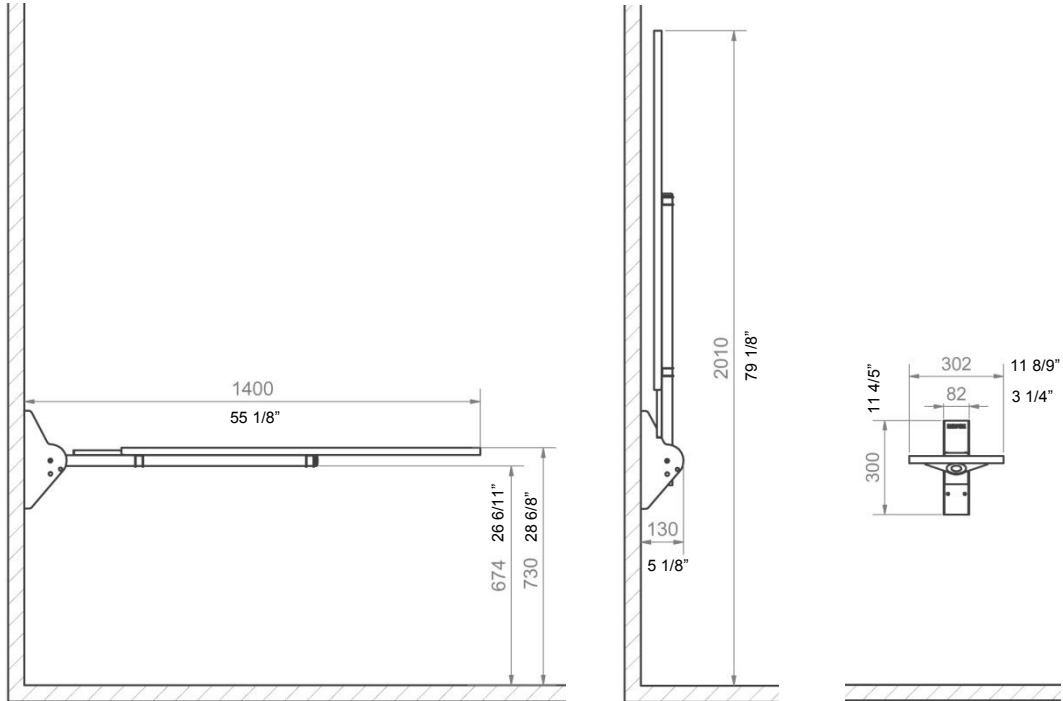
This product has CE marking and has been tested according to the European standard for assistive technology products DS/EN ISO 12182. The product meets the relevant requirements of the Medical Devices Directive No.93/42/EEC and is in the lowest risk class 1.


User manual

Dimension drawings

10-78500 – fixed height

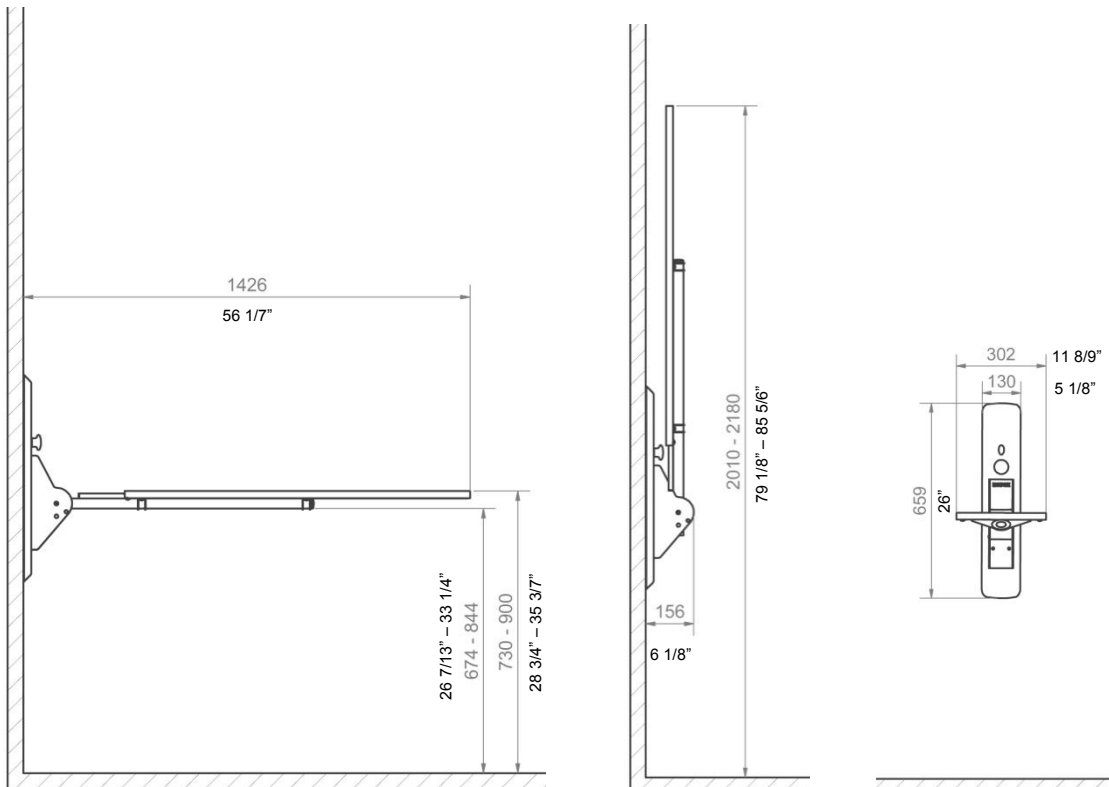10-78500 and 10-78510 – height adjustable

Application

10-78500

The ironing board is wall-mounted. It has no inconvenient legs or braces and is therefore ideal for seated as well as standing users. As it may be folded up against the wall when not in use, persons with functional disabilities may easily use the ironing board without help.

10-78510 (accessory)

The unit is used to adjust the height of the ironing board and has been designed to ensure easy adjustment without the use of tools. The height may be adjusted by pulling the handle and placing the ironing board in the required height. The ironing board is locked automatically when you let go of the handle.

The height may be adjusted 170mm (6 5/7in) at increments of 17mm (4/6in) by pulling the handle.

User manual

Cleaning

The ironing board may get dirty during use and it is very important to clean it according to these instructions.

Simply wipe the Ropox Ironing Board and Height Adjustment Unit with a wrung cloth with lukewarm water and a customary cleaning agent. Do not use etching/grinding liquids or grinding cloths, brushes or sponges.

Disinfection of the product may be made using customary disinfectants.

Label

Each product has a label of its own. This label contains important information to be stated in connection with inquiries to the manufacturer.

Symbols:

The user manual must be read prior to use.

Waste must be sorted – if possible, for recycling.

Ropox is the manufacturer of this product.

CE marking. This product meets the requirements for *General Safety and Functionality* of the relevant EU Directives and Standards.

Safety in use

- Ropox Ironing Board and Ropox Height Adjustment Unit for Ironing Board should only be operated by persons who have read and understood these instructions.
- Always use the ironing board in a manner excluding the risk of damage to persons or property. The person operating the ironing board is responsible for avoiding damage or injury.
- Modifications, which may influence the operation or construction of the Ropox Ironing Board, are not allowed.
- Installation, service and repairs should only be carried out by competent personnel.
- During inspections, service or repairs, load must be removed from the ironing board.

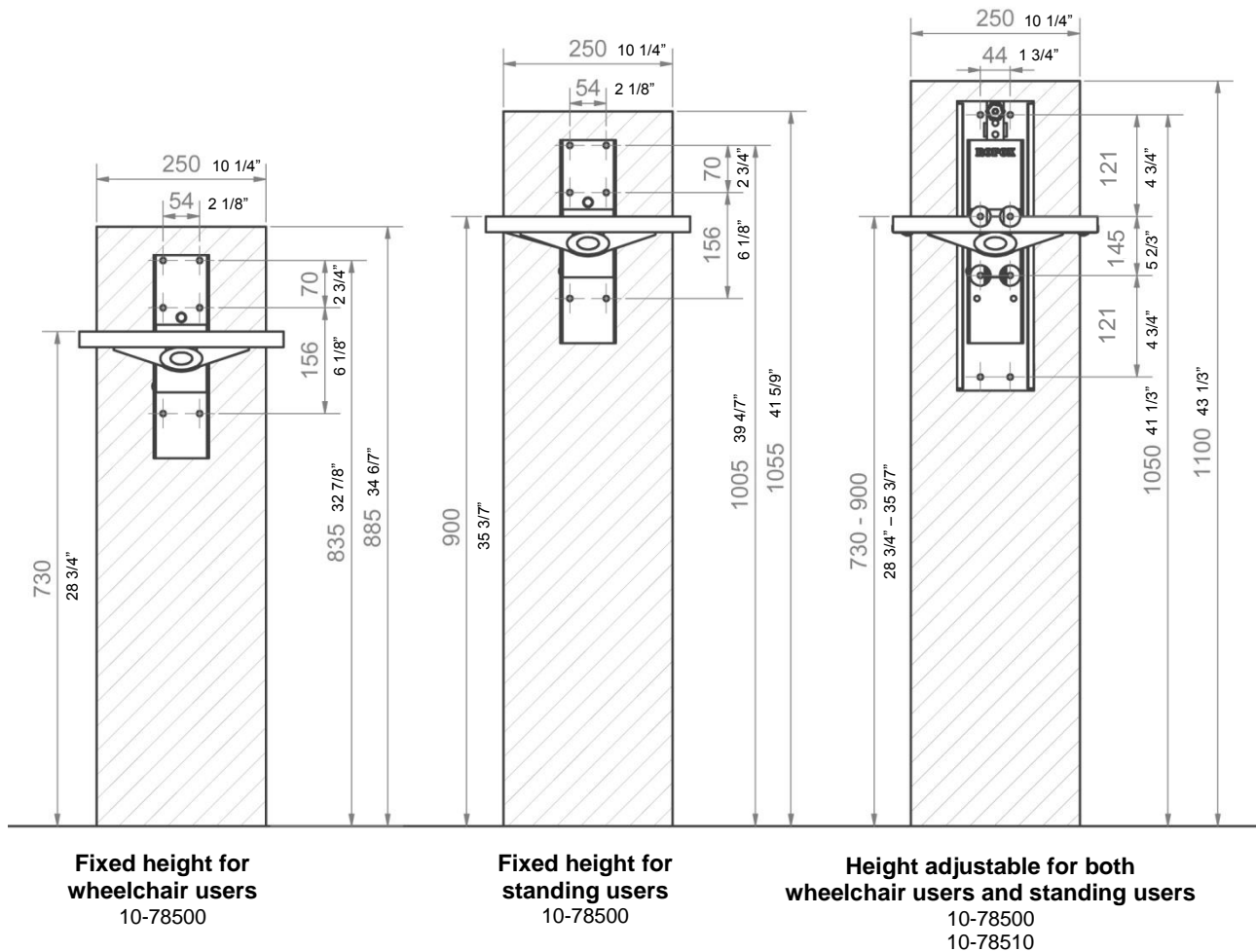
Mounting instructions

Installation information

Location of holes

The location of the ironing board is based on human average measurements.

All dimensions are in mm/inches. If this height of the board is changed, the recommended dimensions must be changed accordingly.

The hatched area is the minimum area for reinforcement of the wall. Reinforcement from floor to ceiling is advisable. The holes are measured on the basis of the upper holes and the location depends on the user and whether a height adjustment unit is installed.

The wall must be made of a material suitable for screw mounting and approved for damp rooms. The product is provided with a set of six or eight (for height adjustment) screws and rawplugs suitable for mounting on a concrete wall. The fitter should always consider the material, condition and strength of the wall and use screws and rawplugs suitable for the specific wall type.

Max load according to DS/EN 12182: 25kg (55lbs)

Max tensile load of upper screws: 20kg (44lbs)

Mounting instructions

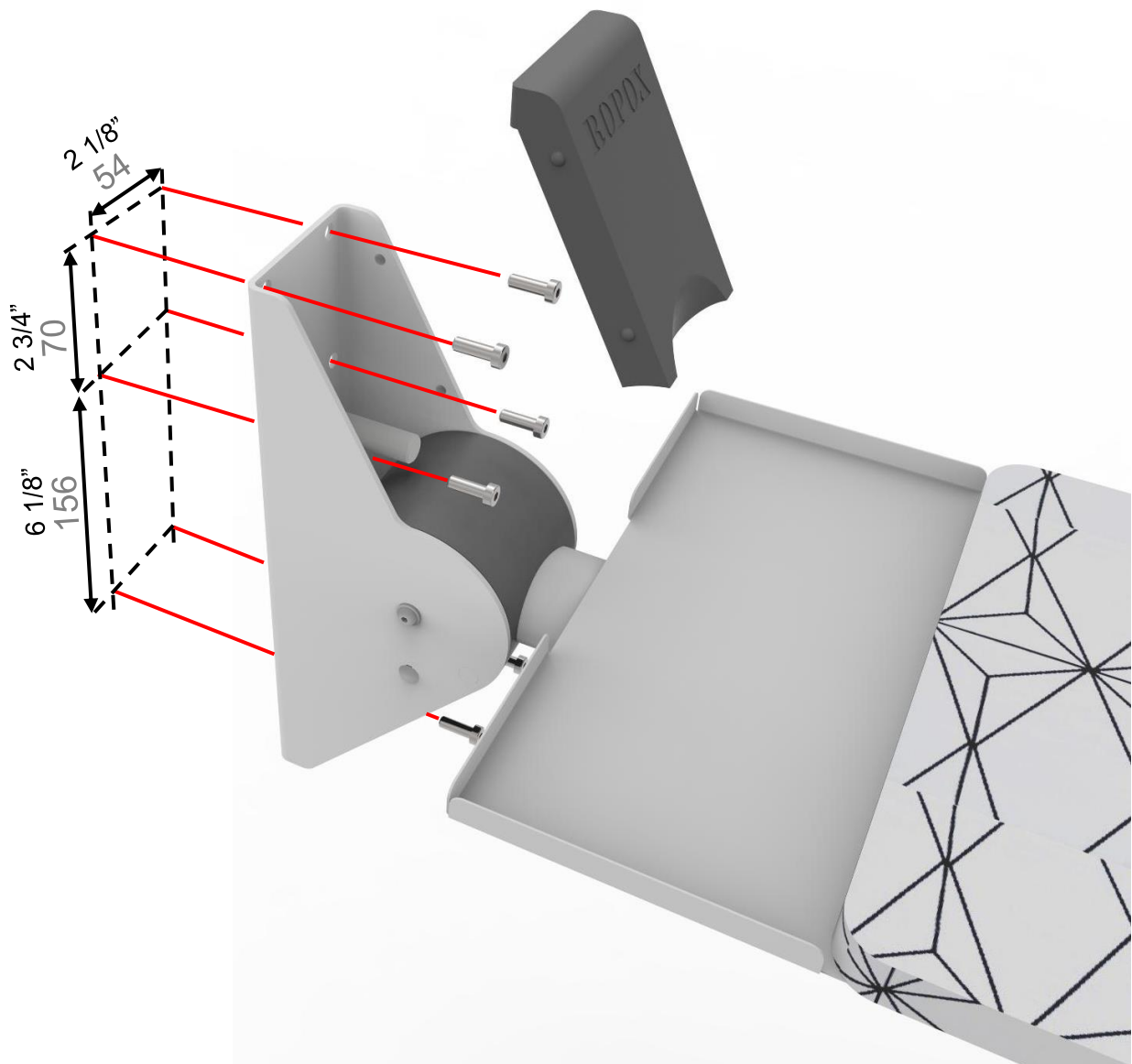
10-78500

Note:

Mounting of screws in the wall must always be made considering the strength and condition of the specific wall. The screws provided are intended as a guide for mounting on a reinforced wall considering the max. screw loads stated in the mounting instructions.

Note:

For location of the upper holes, see installation interfaces

Mounting instructions

10-78500 + 10-78510

Note:

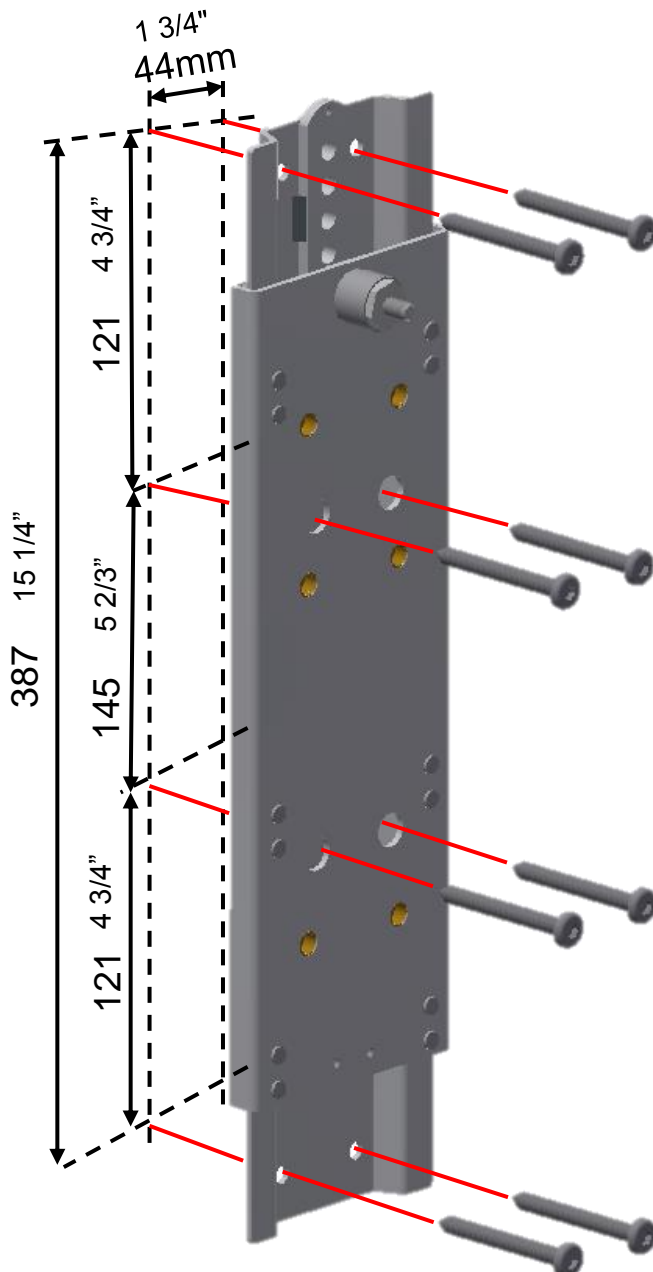
10-78510 is an accessory and can be purchased separately.

Note:

Mounting of screws in the wall must always be made considering the strength and condition of the wall. The screws provided are intended as a guide for mounting on a reinforced wall considering the max. screw loads stated in the mounting instructions.


Note:


For location of the upper holes, see installation information.

Recommended:
8 x 7,5x72
8 x rawlplugs

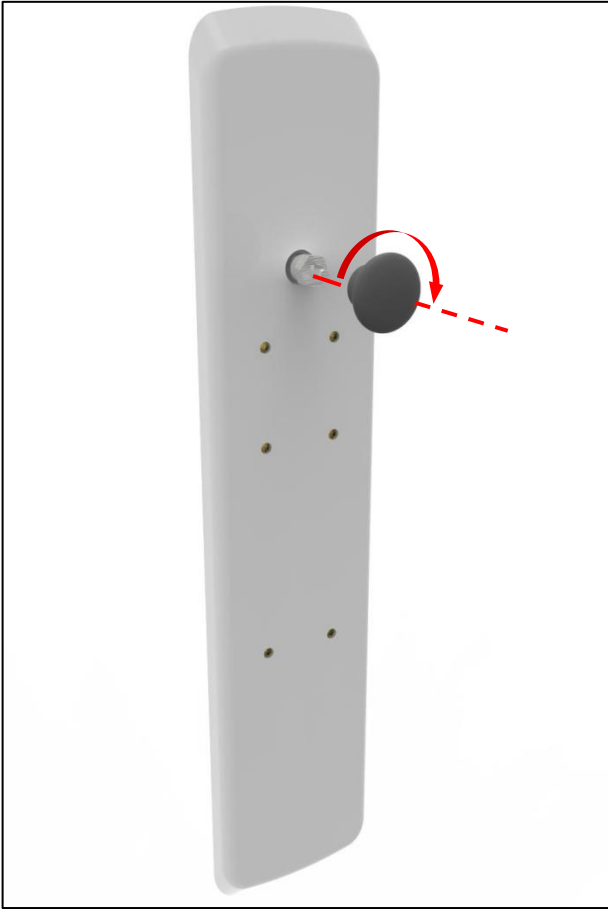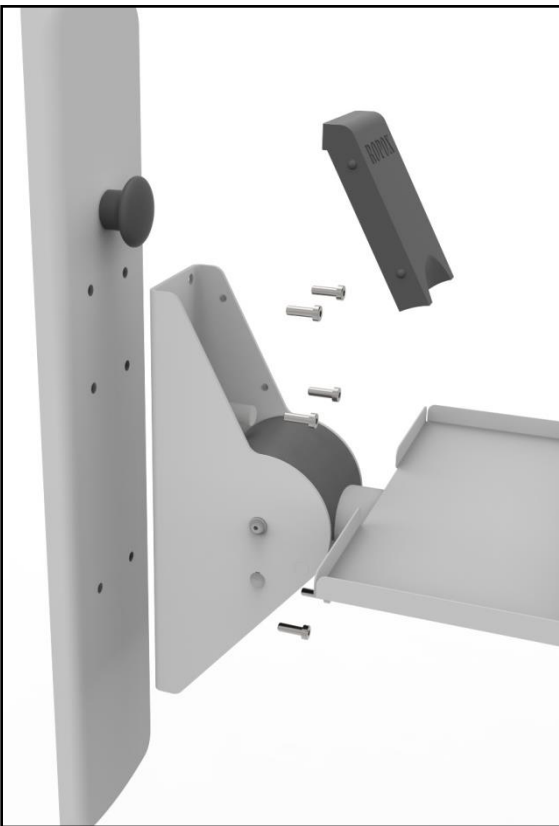Mounting instructions

Service information

Maintenance

Inspections, service and repairs must be carried out by competent personnel

Ropox Ironing Board and Ropox Height Adjustment Unit for Ironing Board are maintenance-free. Moving parts have been lubricated for life. For reasons of safety and reliability, we recommend an annual performance test.

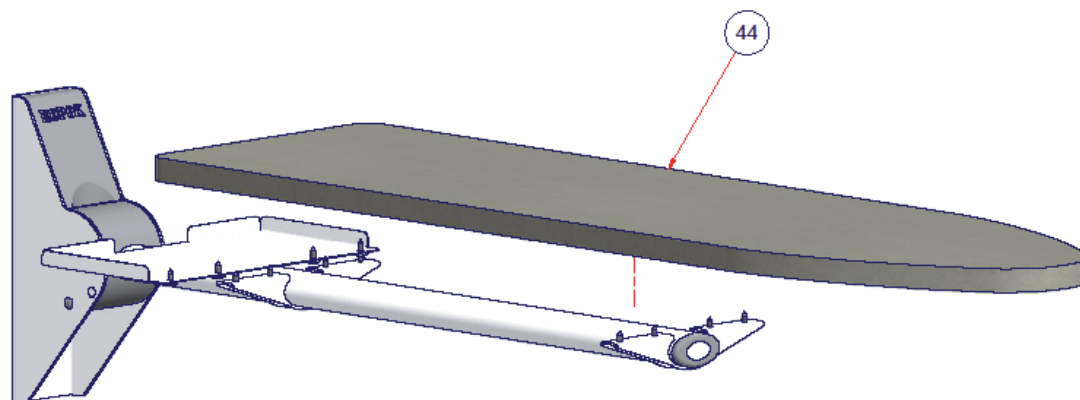
Service inspection

After one month

Check: that bolts and screws have been securely tightened
 that the ironing board can easily be folded up and down.
 that the height adjustment unit can easily be adjusted 120mm (4 $\frac{3}{4}$ ") upwards
 and downwards.


Spare parts

10-78500

Part number	Description	Item number
44	Ironing board assembled	10*78500-059

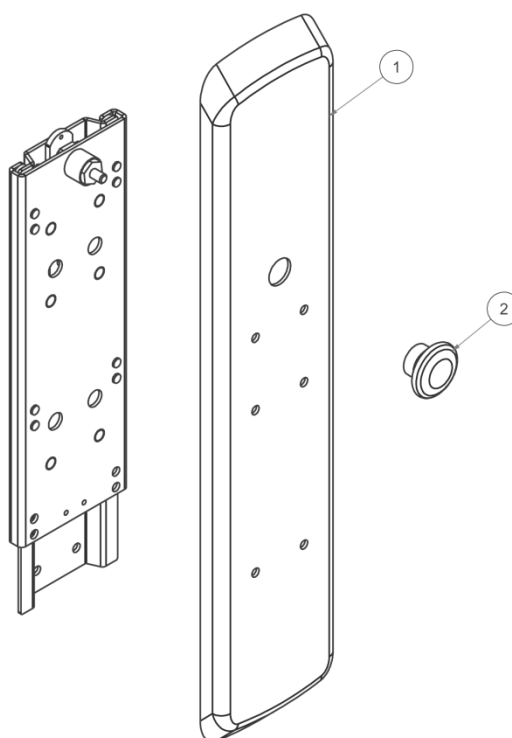10-78510-2

Part number	Description	Item number
2	Knob	40*43043-027
1	Cover plate	40*40907-081

EU – Declaration of Conformity

I, the undersigned, hereby declare that the following products:

Ropox Ironing Board

Item number: 10-78500, Ropox Ironing Board

Ropox Height Adjustment Unit for Ironing Board

Item number: 10-78510, Ropox Height Adjustment Unit for Ironing Board

conform to the following Directives and Standards:


DIRECTIVES

Medical Devices Directive No. 93/42/EEC (risk class 1)

STANDARDS

DS/EN 1041: 2009	Information supplied by the manufacturer of medical devices.
DS/EN 12182: 2012	Assistive devices for persons with functional disabilities – General requirements and test methods.
DS/EN 14971: 2012	Medical devices – The use of risk management in connection with medical devices.
DS/EN 9999: 2011:	Assistive devices for persons with functional disabilities - Classification and terminology.

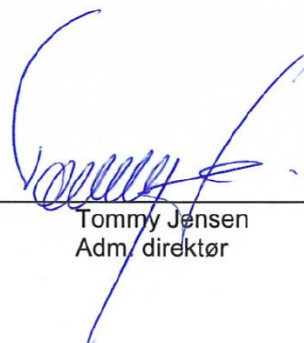Date: 20-06-2016

Tommy Jensen
Adm. direktør

Complaints

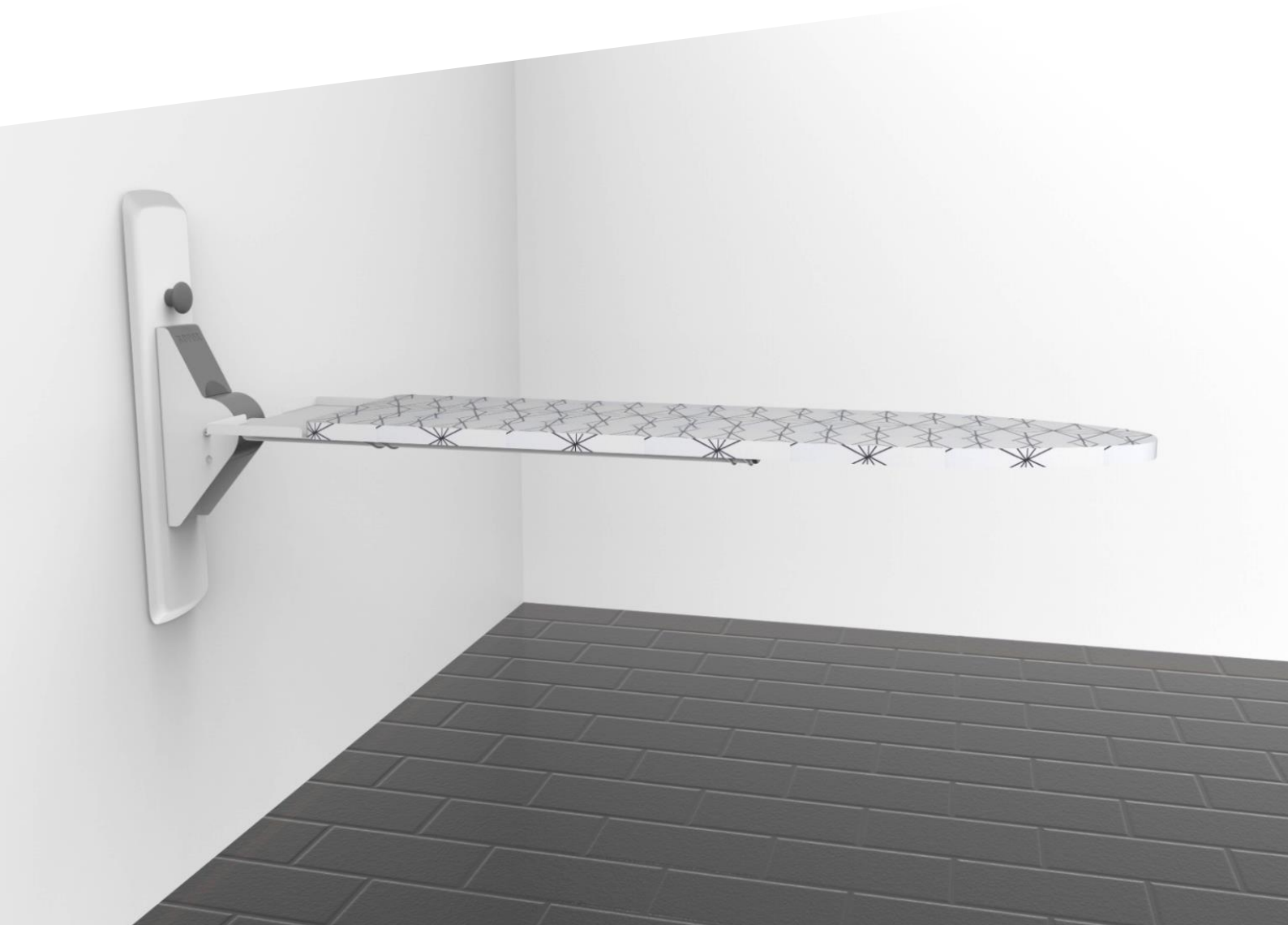
See General Terms of Sale and Delivery on www.ropox.dk

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ROPOX A/S