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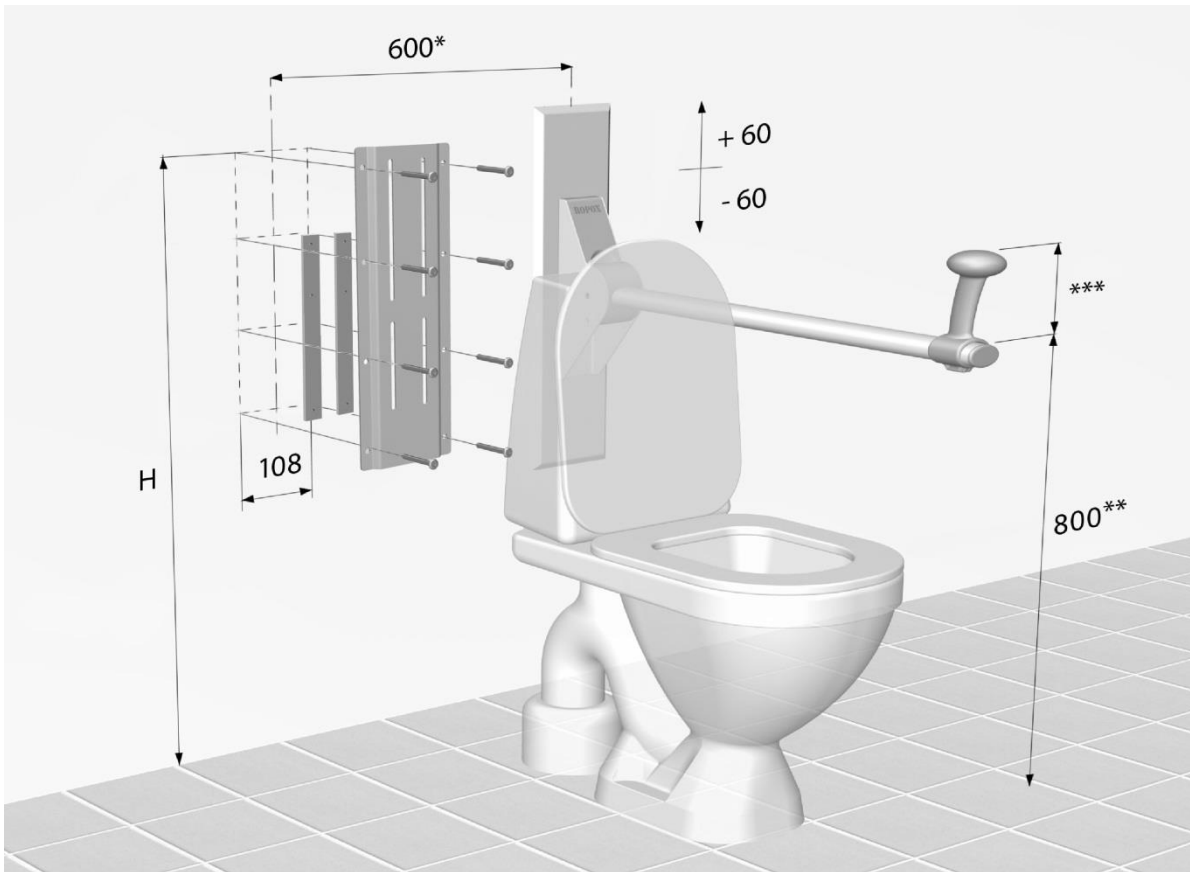
Wall Plate, Wand montage platte

Montageanvisning

Mounting instructions

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Ropox Straight, DS 3028

Alle mål i mm

All measurements in mm

Alle Abmessungen in mm

* I henhold til DS 3028

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* According to the DS 3028

** With various users a height of 800mm is recommended.

In this case H = 980mm

*** If Get up/sit down fitting is used it is recommended that the top is placed 800mm above floor. **In this case H = 850mm**

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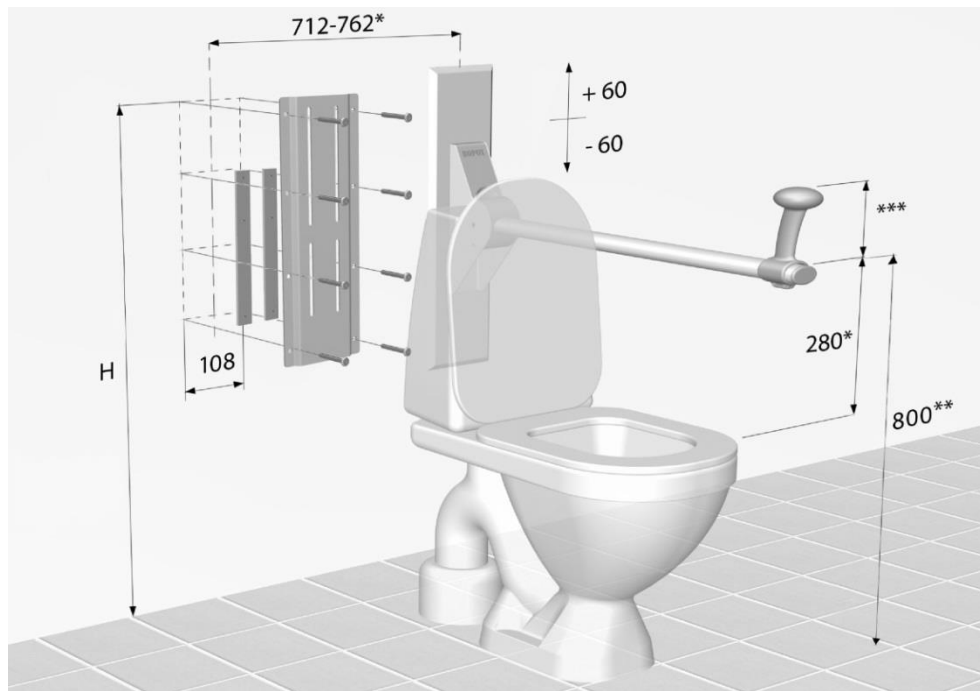
* According to the DIN 18040-1

** With various users a height of 800mm is recommended.

In this case H = 980mm

*** If Get up/sit down fitting is used it is recommended that the top of this is placed in 800mm above the floor. **In this case H = 850mm**

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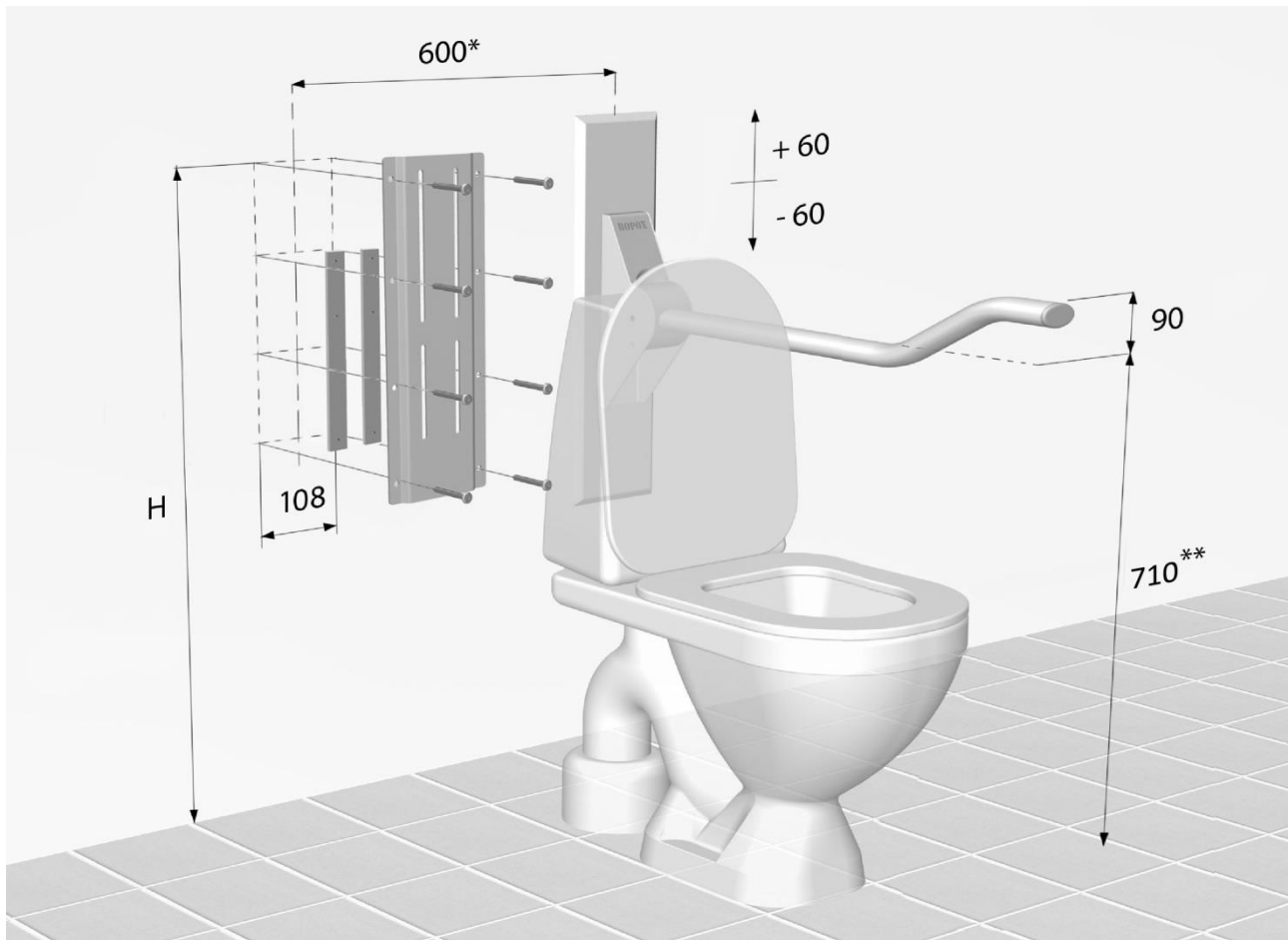
Dette betyder at H = 890mm

* According to the DS 3028

** With various users a height of 710mm is recommended.

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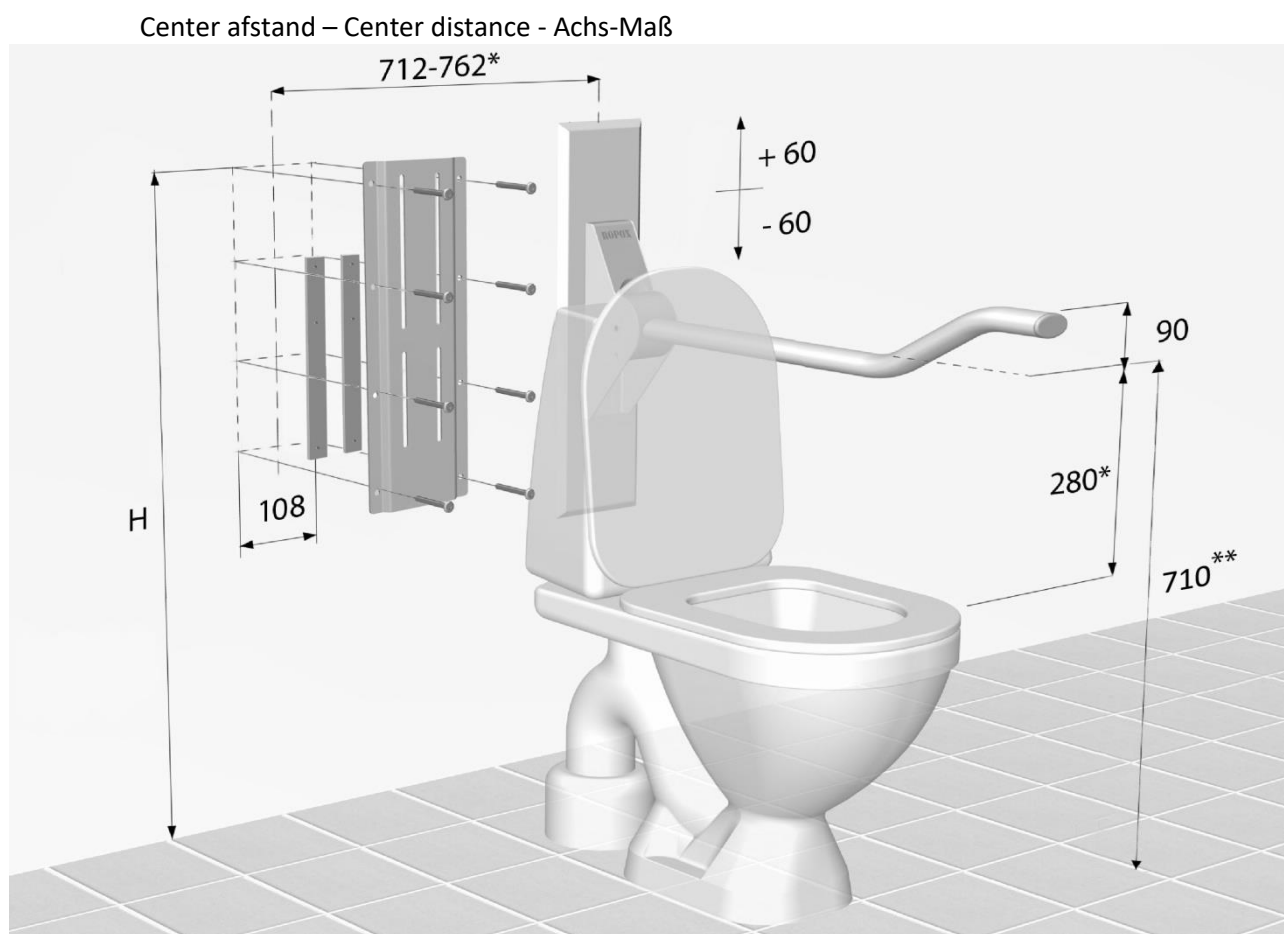
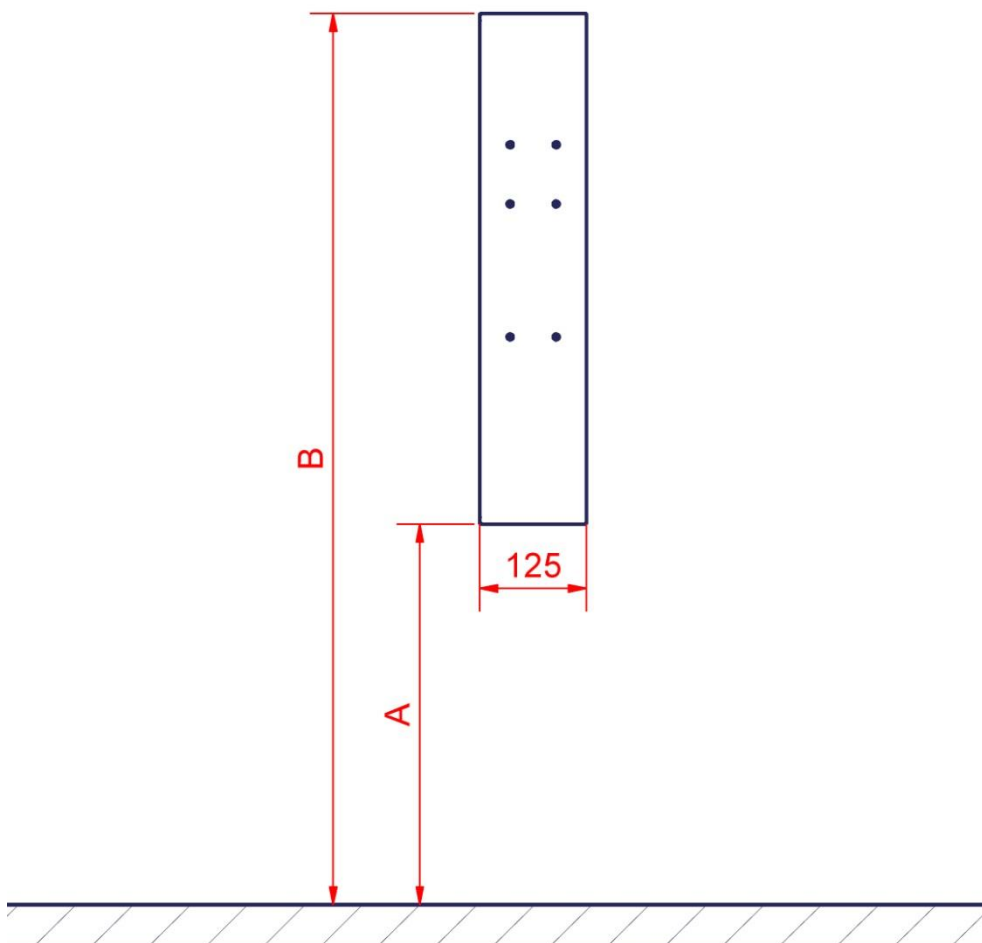
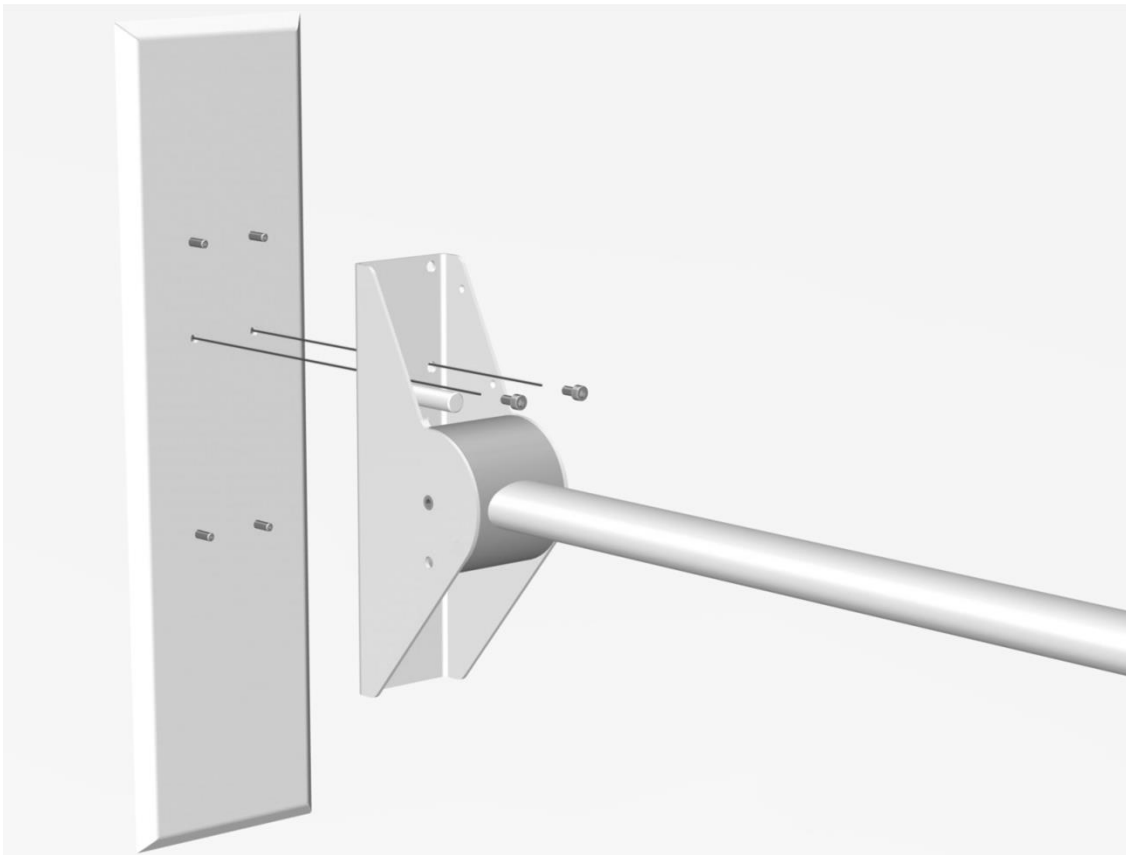
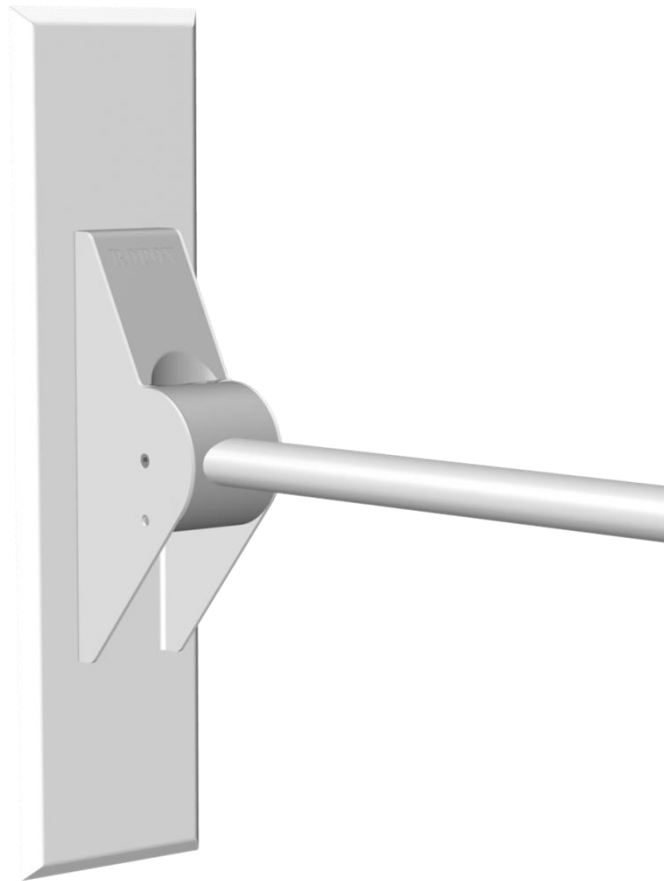
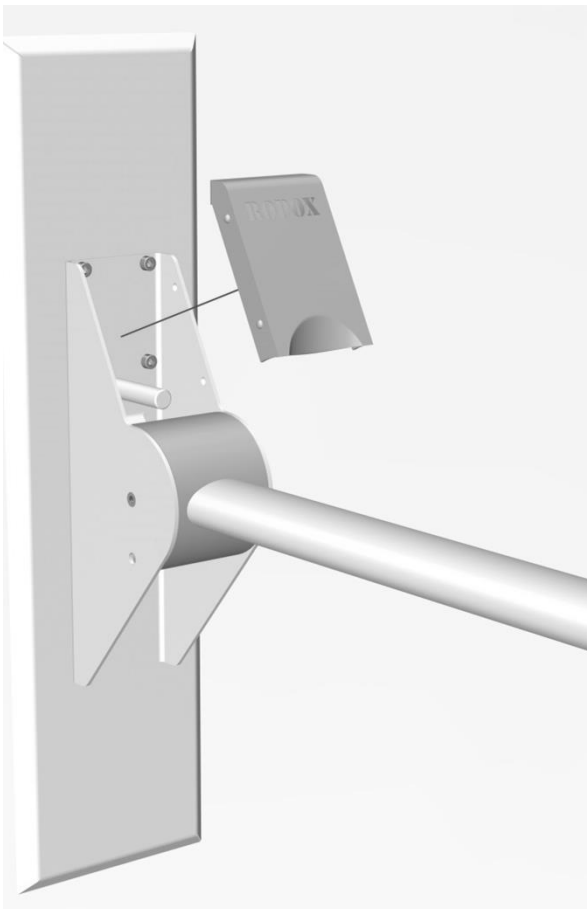
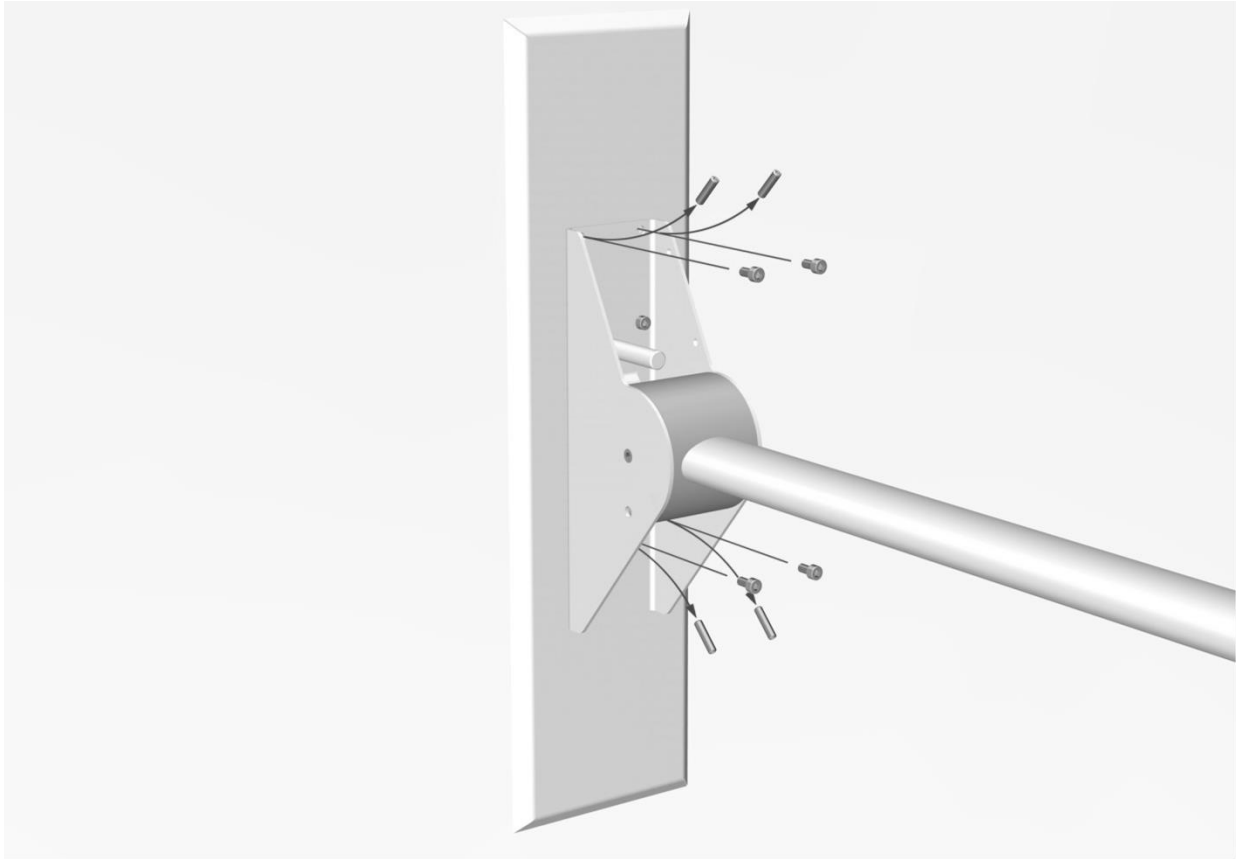
Wall mounting

The screws and rawlplugs included are for mounting on a concrete wall. The installer must always examine the material, condition and strength of the wall and use appropriate screws and rawlplugs.

Maximum load in the top screws are 72kg.
(for Ropox Toilet Support Arms 40-40110, 40-40115, 40-40120 og 40-40125)

Maximum load in the top screws are 171kg.
(for Ropox Toilet Support Arms 40-40140 og 40-40145)

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Maintenance

The toilet supports are maintenance free. For safety reasons it is recommendable to check all mounting screws and moving parts to be sure all is tightened ok, and that all attachment to the wall is in correct and safe condition.

Cleaning

To clean this product, do not use a high-pressure washer. This product is not machine washable.

Simply use a soft cloth or sponge with soapy water (pH 7) of max 60°C. Always clean this product after use to prevent dirt from sticking to the surface.

Important:

Do not use scouring powder or other abrasives as they may scratch the surface. Also avoid the use of cleaning utensils with sharp edges. Never use strong mineral acids or alkali.

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Se Generelle salgs- og leveringsbetingelser på www.ropox.dk

 **Complaints**

See General Terms of Sale and Delivery on www.ropox.com

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ROPOX A/S