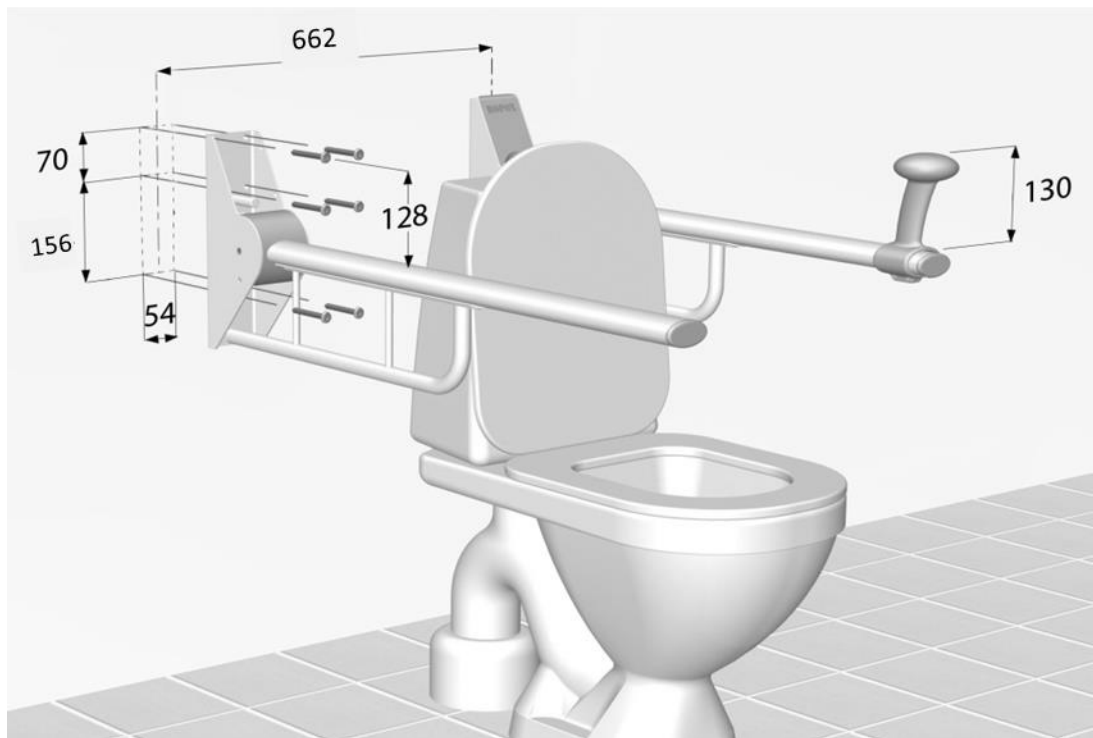
## Ropox Toiletstøtte Straight Bariatric, 40-40140

/ Ropox Toilet Support Arms Straight Bariatric

/ Ropox Toilettenstütze Straight Bariatric

Montageanvisning / Mounting instruction / Montageanleitung

 I henhold til / according to DS 3028 (Danish Standard)

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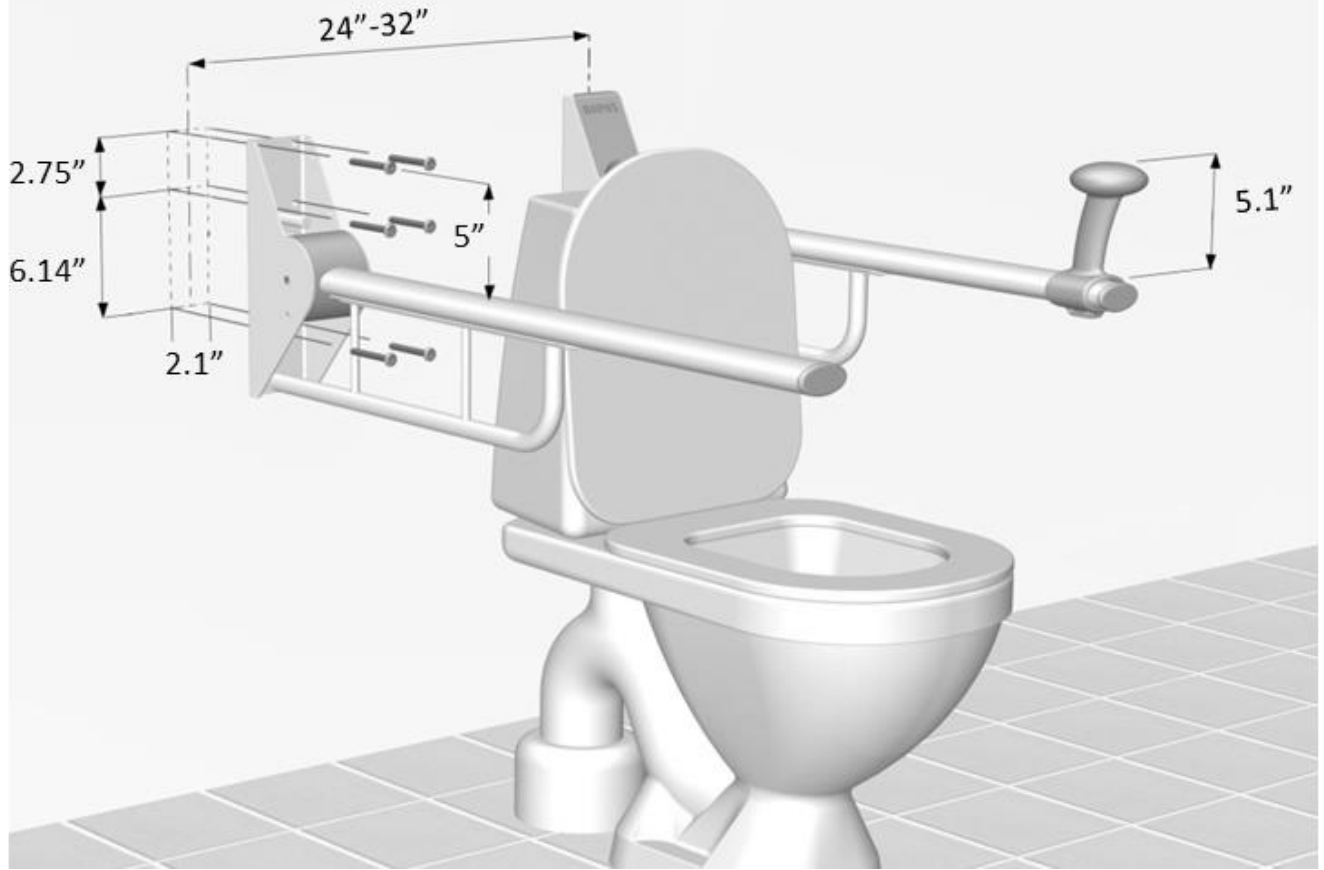
Please check with the user for individual demands which could affect the mounting position. Also make sure that national and/or individual state statutes are fulfilled


Please find attached mounting template showing position of the 6 holes for easy installation  
**User heights and installation heights (upper holes in mounting template/bracket)**

User height (recommendation)	Installation height (upper holes)	Situation
35.5in (top of support arm)	36.5in	Ropox Straight Toilet Support Arm
26.4in (top of support arm) 35.5in (top of Get up/sit down)	31.4in	Ropox Straight Toilet Support Arm + Get up/sit down fitting
<p>If any other specific user height is required, simply position the 2 upper holes of the bracket (or the mounting template) 5in above the required height of the support arm.</p> <p><u>Example:</u> Required user height is 30.5in            Installation height = 30.5in + 5in = 35.5in</p>		

Always read and understand the mounting instruction and safety in use before mounting this product.

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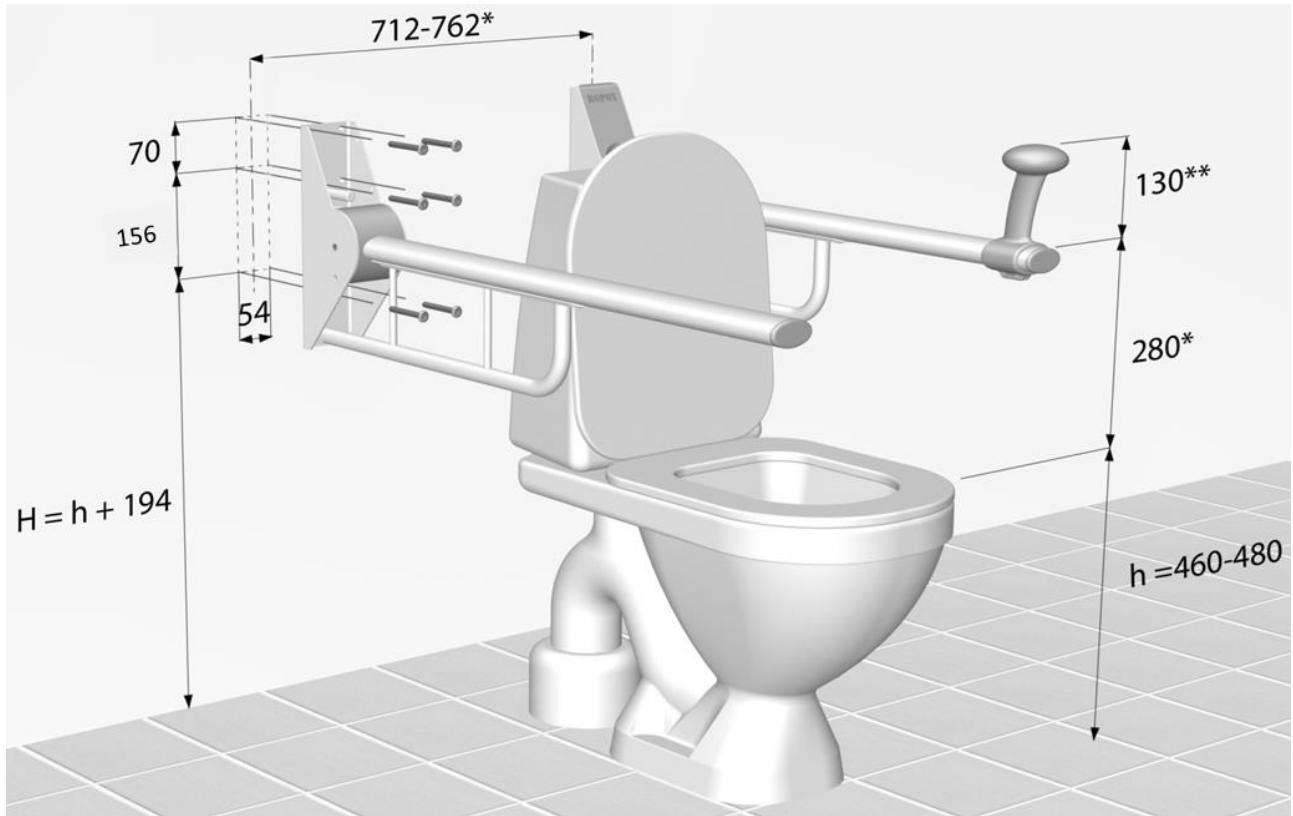
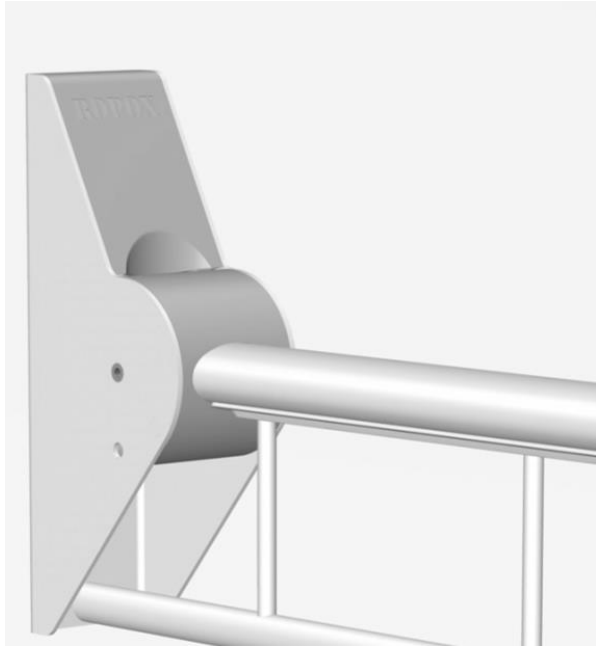
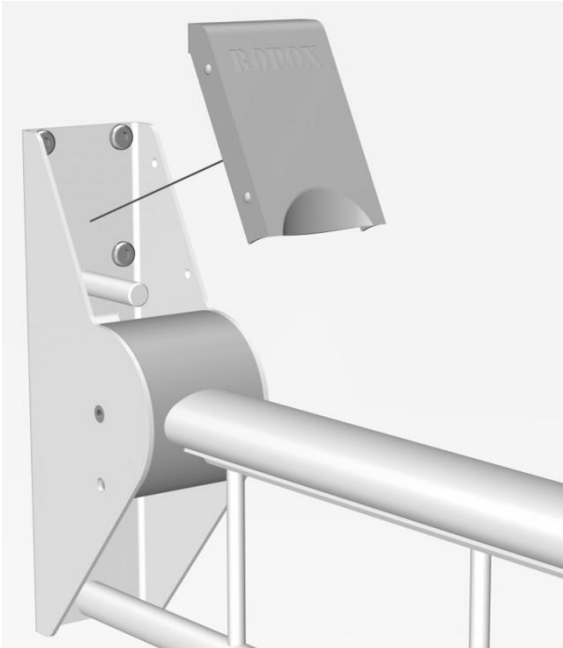
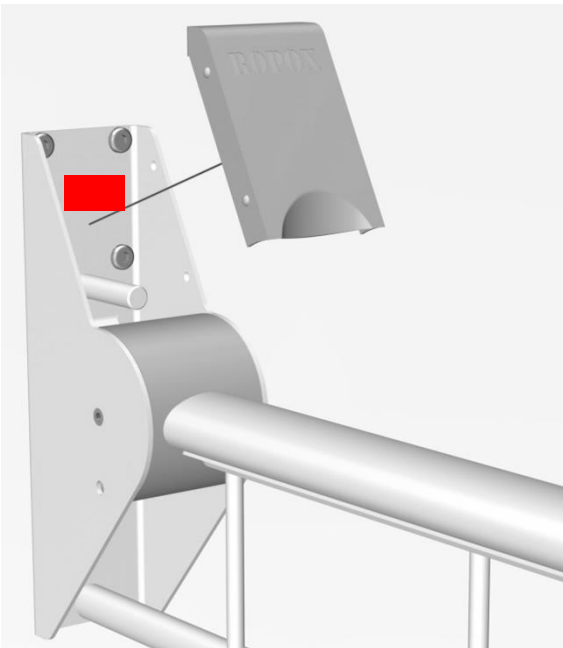
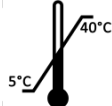
	<p>This product is CE-marked in accordance with:</p> <p>European Medical Device Directive 93/42/EEC, including amendments incorporated in Directive 2007/47/EEC.</p> <p>European Medical Device Regulation (EU)2017/745</p> <p>Council Directive 2011/65/EU, RoHS</p>
	<p>Manufacturer name and address</p>
	<p>Date of production</p>
	<p>Stock number</p>
	<p>Serial number</p>
	<p>Consult manual before use</p>
	<p>Consult manual for important safety related information, warnings and safety precautions.</p>
	<p>Do not dispose as unsorted municipal waste. Product must be returned to a designated recycling station.</p>
	<p>Operating temperatures may not exceed +5 ° to +40 °C  Transport and storage temperatures may not exceed -5 °C to + 40 °C</p>

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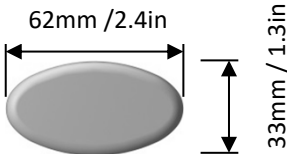## Safety in use

- Do not operate the toilet support in case of defects or damage.
- Do not overload the toilet support.
- Modifications, which may influence the operation or construction of the toilet support, are not allowed.
- Installation, service and repairs must be carried out by competent personnel.
- If the toilet support has not been assembled according to these mounting instructions the right to complain may become void.
- Only use Ropox original spare parts as replacement parts. If other spare parts are used, the right to complain may become void.

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## Wall mounting

The screws and rawlplugs included are for mounting on a concrete wall. The installer must always examine the material, condition and strength of the wall and use appropriate screws and rawlplugs.

To comply with DS/ISO 17966 the maximum user load is set to 385kg/847lbs.

Maximum pull in the top screws are 160kg/352lbs.

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## Intended use

The Toilet support arms makes people with reduced mobility more self-reliant. The Toilet support arms provide the user with stable support and something to lean on when getting up or sitting down and for some people, it makes them able to use the toilet on their own. The Toilet Support Arms can be adjusted in height by 120 mm by using a wall-mounted, height adjustable fitting. They can either be adjusted manually with an allen wrench or they can be fitted with a mechanism that allows a user or carer to adjust the height with one hand.

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## Maintenance

The toilet supports are maintenance free. For safety reasons it is recommendable to check all mounting screws and moving parts to be sure all is tightened ok, and that all attachment to the wall is in correct and safe condition.

## Cleaning

To clean this product, do not use a high-pressure washer. This product is not machine washable. Simply use a soft cloth or sponge with soapy water (pH 7) of max 60°C. Always clean this product after use to prevent dirt from sticking to the surface.

Important:

Do not use scouring powder or other abrasives as they may scratch the surface. Also avoid the use of cleaning utensils with sharp edges. Never use strong mineral acids or alkali.

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## Complaints

See General Terms of Sale and Delivery on [www.ropox.com](http://www.ropox.com)

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## ROPOX A/S